

INSTRUCTION FOR USE

INTENDED FOR USE

This product to be used with insulin injector is a single use sterile insulin pen needle to be used for injecting insulin to patient with diabetes.

USAGE METHOD

1. Remove pen needle seal.
2. Screw pen needle onto pen, by pressing straight down using the outer protective cap (Do not click on at an angle).
3. Remove the outer protective cap by pulling straight off save it.
4. Remove the inner protective cap.
5. Penetrate skin quickly and inject slowly.
6. Carefully replace the outer protective cap (if self-injecting only).
7. Unscrew the pen needle and safely dispose of it according to your physician's or pharmacist's advice.

WARNINGS

Do not allow children to use this device without parental supervision.

Avoid infection

1. Do not use the pen needle if the seal is damaged or has been opened by someone other than you.
2. Do not use the pen needle after the expiration date on the packaging has expired.
3. Before each injection: clean the injection site thoroughly according to your physician's instruction.
4. Use each pen needle only once.
5. Do not re-use, clean and/or re-sterilize pen needles.
6. Carefully replace the outer protective cap on the used pen needle and dispose of it immediately after self-injection. Follow your physician's or pharmacist's advice for proper disposal in your area.
7. If these instructions are not followed, there is a risk of infection.
8. Seek medical attention if you have any health concern. Never Handle someone else's pen.
9. To avoid accidental needle sticks, which can transmit serious infections, never recap, remove, or destroy a pen needle that has been used by someone else.

Get the correct dosage

1. Consult your physician for the right needle length for you.
2. Prime your pen according to your pen's instructions for use before each injection to ensure that the needle allows free flow of medication.
3. Inject slowly according to your physician's advice and pen's instructions for use.
4. Check after each injection whether medication drips from the pen needle, pen, or injection site.
5. If the medication drips, follow your physician's instructions regarding the handling of potential under-dosage.
6. Seek medical attention if you have any health concern. Avoid bending or breaking needle.
7. Do not change the direction of the pen and pen needle during Insertion into or withdrawal from the skin.
8. Seek immediate medical attention if a broken needle becomes stuck in the body or disappears under the skin.





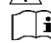


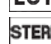




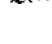

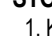
A SAFE AND COMFORTABLE INJECTION

1. Change the injection site regularly in accordance with your physician's advice. If this is not done, tissue hardening may result.
2. Inject medication at room temperature.
3. Make sure no air bubbles remain in the pen before injection.
4. Wait until topical alcohol (if used) has evaporated completely before injection.
5. Keep muscles in the injection area relaxed when injecting.
6. Penetrate the skin quickly.
7. Do not change the direction of the needle during insertion into or withdrawal from the skin.
8. Do not re-use pen needles.

COMPATIBLE INJECTORS

INJECTION PEN NAME	INJECTION PEN NAME
Apidra SoloStar	NovoLogMix70/30 FlexPen
Lantus SoloStar	Victoza Pen
Humalog KwikPen	NovoPen 3
HumalogMix 50/50 KwikPen	NovoPen 4
HumalogMix 75/25 KwikPen	NovoPen Echo
Humulin N Pen	NvoPen Junior
Humulin 70/30 Pen	Byetta Pen 5mcg
HumaPen Luxura	Byetta Pen 10mcg
Forteo	SymlinPen 60
Levemir FlexPen	SymlinPen 120
NovoLog FlexPen	AutoPen

Description of the symbol on the Label for:

-  "Manufacturer"
-  "Authorised Representative in the EC"
-  "Use by"
-  "Manufacture Date"
-  "Attention, See Instruction for use"
-  "Consult Instruction for use"
-  "Do not use if package is damaged"
-  "Do not reuse"
-  "Batch code"
-  "Ethylene oxide gas sterilization"
-  "Keep away from sunlight"
-  "Keep dry"
-  "Room Temperature: 1~30 °C"
-  "Fragile"
-  "Use no hook"

STORAGE KEEP CONDITIONS

1. Keep dry place.
2. Keep away the sunlight.
3. Keep temperature under 30 degree.

 Interkotra GmbH

Kurfuerstenplatz 34 D-60486

Frankfurt am Main Germany

Phone : +49 (0)69-778914

TAE CHANG INDUSTRIAL CO., LTD.

127, Sanho-daero, Gumi-si, Gyeongsangbuk-do, Korea

IMPORTED BY TERUMO

DISTRIBUTED BY TERUMO